## CERRAHİ ALET BAKIM KULLANIM KLAVUZU

www.bahadir.com E-mail: info@bahadir.com.tr

1

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2

### Product Specifications

This product is manufactured of stainless steel, in compliance with 93/42 EEC directive requirements about medical devices.

### Intended Usage

Surgical hand instrument are used for cutting, puncturing, sawing, unting, squeezing, pulling, holding or similar surgical operations during, surgical operations.

### Warnings

Authorized personnel that have medical science education should use surgical instruments according to their intended usage. Instruments that are subject to corrosion should not be used. Instruments should be sterilized before usage. Newly purchased instruments should not contain any chloride ions. pH value of the cleaning agent should be 7. Water used for cleaning and rinsing of the instruments should be pure water (demineralized water).

### Usage of Newly Purchased Instruments

Newly purchased instruments should be taken out of their package and kept in dry places. They should not be kept near to chemical substances that produce corrosive vapor. If exist all the packing and bandage should be removed and instruments should be cleaned before sterilization.

### Maintenance and Cleaning

Cleaning of the instruments should be carried out with suitable chemicals under consideration of contamination risks, after operations. Suggested dosage values of the chemical materials should be followed exactly according to instructions supplied by manufacture+s. Metal brushes or metal tools should not be used on the instruments during cleaning.

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3

Instruments with joint should be disassembled before cleaning. The utmost attention should be showed to delicate instruments such as micro surgery instruments, endoscopes etc. Instruments should be rinsed and dried and should be lubricated with an autoclavable paraffin-oil based lubricant, after after cleaning. Spot formation, corrosion formation and function control should be carried out and defected instruments should be separated from others. Instruments that have a locking mechanism (ratchet) should be locked in to first level of locking position. Instruments should be placed in to wire baskets or on to silicone mats carefully. Users and instruments should not be subject to any harm during cleaning and placement of the instruments that are used for cutting or puncturing.

### Transportation and Storage

Used instruments should be transported under consideration of contamination risk and institutions' instructions should be followed during this procedure. Instruments should be transported, stored and distributed by authorized personnel, according to institutions' instructions. Instruments should be kept in controlled environments that is free of dust and insects, places that does not get direct sun light and places that have air conditioning systems. Access of unauthorized personnel to these rooms should also be kept under control.

### Service Life

There is no standard service life for the instruments. As long as they are used according to their intended usage, as long as storage and maintenance instructions are followed, they can be used for years.

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**FACTORY AND HEAD OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Organize Sanayi 55300  
SAMSUN/TÜRKİYE  
Tel: 90 (362) 266 70 01 (PBX)  
Fax: 90 (362) 266 70 05

**MARMARA REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Miliet Cd. Özbek Süleyman Efendi Sk.  
Aras Apt.No.8/21  
Fındıkzade / İSTANBUL / TR  
Tel: 90 (212) 632 80 34 - 632 80 35  
Fax: 90 (212) 588 19 76

**EGE REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Mustafa Kemal Paşa Cad.  
Mutlu Apt.No.141 Bomova / İZMİR / TR  
Tel: 90 (232) 343 85 45 - 348 85 46  
Fax: 90 (232) 343 85 48

**AKDENİZ REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Reşatbey Mh.Avukat M.Eroğlu Sk.  
Berköz Apt.No.2 D.13 Kat.2 ADANA/TR  
Tel: 90 (322) 458 02 72  
Fax: 90 (322) 458 02 92

**KARADENİZ REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
İslikial Cad. Kaptanaga Sk.  
Ali Çepni İshani No:4/17 SAMSUN / TR  
Tel: 90 (362) 228 65 00 - 228 65 01  
Fax: 90 (362) 228 09 23

**İç Anadolu Bölgesi**  
BAHADIR TIBBİ ALETLER  
Talat Paşa Bulvarı  
No:113/13 Kat:1 ANKARA / TR  
Tel: 90 (312) 310 36 06 - 310 36 07  
Fax: 90 (312) 310 36 08

**EUROPEAN REPRESENTATIVE**  
ERMIS CHIR INSTRUMENTE  
Mohringer Strabe 3,78532  
Tutlingen/GERMANY  
Phone: +49.7461.16 03 28  
Fax: +49.7461.16 05 98

**FRANCE**  
S.A.R.L. SAMMEX  
89, Rue Ernest Renan  
F-51100 Reims/FRANCE  
Phone: +33.326.02.64.30  
Fax: +33.326.02.21.56

**U.S.A.**  
BAHADIR-USA CO.  
Web: www.bahadir-usa.com  
e-mail: info@bahadir-usa.com

**AUSTRALIA**  
TIP TOP FPE  
695 Sydney Rd. Couburg 3058  
Victoria/AUSTRALIA  
Phone: +613 9350 2377  
Fax: +613 9350 2677

**İÇ ANADOLU REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Talat Paşa Bulvarı  
No:113/13 Kat:1ANKARA / TR  
Tel: 90 (312) 310 36 06 - 310 36 07  
Fax: 90 (312) 310 36 08

**EUROPEAN REPRESENTATIVE**  
M.ERMIS Import -Export  
Mohringer Straße 1-3  
78532 Tutlingen/GERMANY  
Phone:+49.7461.16 05 28  
Fax: +49.7461.16 05 98

**FRANCE AGENT**  
SARL SAMMEX  
89, rue Ernest Renan  
F-51100 Reims/FRANCE  
Phone: +33.326.02.64.30  
Fax: +33.326.02.21.56

**BAHADIR USA CROP.**  
428 Peninsula Blvd.  
Hempstead N.Y.11550  
Tel: +1516 565 3607  
Fax: +1516 565 3794

**AUSTRALIA AGENT**  
TIP TOP FDE  
695 Sydney Rd. Couburg 3058  
Victoria/AUSTRALIA  
Phone:+613 9350 2377  
Fax: +613 9350 2677

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4

### OFİSLER

**FACTORY AND HEAD OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Organize Sanayi Bölgesi  
SAMSUN/TÜRKİYE  
Tel: 90 (362) 266 70 01 (PBX)  
Fax: 90 (362) 266 70 05

**MARMARA REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Milli Cdt. Çzcek Süleyman Ekinli Sk.  
Aras Apt.No.8/2 Fındıklıca / İSTANBUL / TR  
Tel: 90 (212) 632 80 34 - 632 80 35  
Fax: 90 (212) 688 19 76

**EGE REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Mustafa Kemal Paşa Cad.  
Mutlu Apt.No.141 Bornova / İZMİR / TR  
Tel: 90 (322) 343 85 45 - 343 85 46  
Fax: 90 (322) 343 85 48

**AKDENİZ REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Reşatbey Mh.Avvukat M.Eroğlu Sk.  
Berkoz Apt.No.2 D.13 Kat:2 ADANA / TR  
Tel: 90 (322) 458 02 72  
Fax: 90 (322) 458 02 92

**KARADENİZ REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
İstiklal Cad. Kaptanpaşa Sk.  
Ali Cepni İğarı No.4/17 SAMSUN / TR  
Tel: 90 (362) 228 65 00 - 228 65 01  
Fax: 90 (362) 228 09 23

**İÇ ANADOLU REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Talat Paşa Suluvarı  
No.113/13 Kat:1ANKARA / TR  
Tel: 90 (312) 310 36 06 - 310 36 07  
Fax: 90 (312) 310 36 08

**EUROPEAN REPRESENTATIVE**  
MEREMIS Import-Export  
Möhringer Straße 1-3  
78532 Tuttlingen/GERMANY  
Phone: +49 7461 16 05 28  
Fax: +49 7461 16 05 98

**FRANCE AGENT**  
SARL SAMMEX  
89, rue Ernest Renan  
F-51100 Reims/FRANCE  
Phone: +33 328 02 84 30  
Fax: +33 328 02 21 56

**BAHADIR USA CORP.**  
428 Peninsula Blvd.  
Hempstead, N.Y 11550  
Tel: +1 516 665 3607  
Fax: +1 516 565 3794

**AUSTRIA AGENT**  
TIP TOP FDE  
695 Sydney Rd. Coburg 3058  
Victoria/AUSTRALIA  
Phone:+613 9350 2377  
Fax: +613 9350 2677

Doc. No. : KK-02 / 10.08.2005 / Rev: 00

8

# INSTRUCTION OF USE FOR BAHADIR ALUMINUM CONTAINERS

## CONTENT

Validity .....	1
Description .....	2
Filter system .....	2
Control Before Usage.....	3
Safety Seal.....	3
Internal packing.....	4
Sterilization operational limits .....	4
Placing into Sterilizers.....	5
cards/indicators .....	5
After sterilization .....	5
Storage/Transportation .....	5
Special Cases.....	6
Aseptic presentation .....	6
Cleaning and disinfection .....	6

## VALIDITY

These instructions of use are valid for all BAHADIR Aluminum Sterilizing Containers and accessories, sold by BAHADIR MEDICAL INSTRUMENTS A.Ş. under brand BAHADIR BAHADIR Aluminum Sterilizing Containers and accessories are incompatible with its current catalog. BAHADIR declares that above mentioned products are manufactured in its Quality Management System which has been created in compliance with TS EN ISO 9001:2000 and DIN EN ISO 13485:2003 and under scopes of the standards declares own responsibility.

## DESCRIPTION

Bahadır Aluminum Containers are reusable, metal, sterilization containers. They are designed for holding operating room instruments and/or textiles during vacuum-steam sterilization procedures and for maintaining sterility during storage and transport under proper hospital conditions. (EN 285, EN 868-1, EN 868-8). Sterilization containers consist of three main parts such lid, bottom and filter retainers. The sterilization containers should be handled by qualified personnel that are trained and instructed about sterilization containers, hospital hygiene and sterilization technology, in order to prevent damage to the containers, fasteners, seals and sterilization filters, during usage. This user manual describes important instructions on the proper usage and maintenance of Bahadır Aluminum containers, and possible hazards that could result from failure to observe the instructions. Endoscopes, instruments with lumen, compressed air driven instruments or power systems and canalized instruments should be prepared and sterilized according to manufacturer's instructions. Container lids are offered in six different colors in order to ease identification of the instruments that are used by different departments in hospitals. The colored identification labels that are used with containers provides information about the content and where these instruments are used in.

## FILTER SYSTEM

BAHADIR Aluminum Containers are available with a unperforated base and perforated filter lid or with perforated bottom and lid (can be covered by a unperforated lid). They are designed to be used with single use (disposable) filters or reusable textile filters. It's necessary to use same type of filters during usage of the containers.

In case of use of filters, which are not supplied from BAHADIR, the user must validate the permeability and barrier properties of the filters himself.

Filters: Disposable single use paper sterilization filters have to be changed before each new sterilization.

Long-term textile filters can be used for about 45-50 sterilization cycles. Visually deformed and dirty textile filters should not be used.

Permanent filters (PTFE) can be used for over 1000 sterilization cycle. During storage after sterilization, in order to prevent damage (puncture, tear) to sterilization filters, sharp and pointed objects should not be placed on the containers. BAHADIR recommends usage of safety lids on the containers during transportation and the storage of the containers in order to prevent contamination risks that may be caused by such negative situations.

## CONTROL BEFORE USAGE

During storage, usage of metal sterilization containers is safer than other storage method of steril materials regard to protection against contamination. Like all reusable equipment, however, the BAHADIR Aluminum Containers tough robust also needs to be treated with care in order to ensure that its protective qualities are preserved. The relevant personnel (including delivery and collection services) must therefore be familiar with the correct handling practices. CAUTION: Careless handling or the use of inappropriate chemicals can cause damage on the containers, thereby putting at risk the ability to attain and preserve sterility. BAHADIR Aluminum Containers therefore require regular visual and, if necessary, functional checks. If cautions and the instructions in the user manual is followed, containers may serve for 1000 sterilization cycle and seals may serve for 500 sterilization cycles.

## Undamaged shape:

- Containers must be checked visually before each usage.
- Container bottoms, container lids and the surfaces where the seals sit must be free of dents and visible deformations.
- Do not use any spray, oil or solvents on the lid seals.
- The seal in the inner lid must be completely inserted and undamaged. If any kind of damage detected lids should not be used.
- When the container is closed, tray, lid and locking parts have to be stable. (No "wobble").
- Maintenance and repairs of the sterilizations containers must be carried out by qualified personnel. Do not attempt to carry out repairs on containers, lids, fasteners and seals yourself, in order not to jeopardize the safety in use of the containers.

## Filters and Filter Retainers

These parts must show no visual deformations. These parts must also be checked visually and for their functionality before usage. Filters should cover all the perforation holes, properly.

Filters retainers should function properly when mechanically checked and filter retainers should be easily attachable and detachable. After any accident (such as a container being dropped on the ground), it is essential that the sterile container undergo a thorough check. Make sure that filters and filter retainers are placed in to their places properly. "Click" sound that is heard while placing filter retainers by pressing on them indicates that locking is realized.

## SAFETY SEAL

It is recommended and required by DIN 58953/9 that containers are sealed in such a way as to prevent inadvertent opening of containers and to ensure that it is evident whether or not a container has been opened. BAHADIR Aluminum Containers can be protected by disposable plastic seals ("security seals"), which, once attached, can be opened by breaking only.

## INTERNAL PACKING

We recommend using BAHADIR Aluminum Containers with simple internal packaging (e.g. cloth wraps or drip sheets). These assist the final drying stage, allow a longer storage period according to DIN 58953/9, and makes aseptic presentation of the sterile goods possible. The size of the cloth wraps should be calculated so that when they are unfolded all the external walls of the container can be covered. As an alternative to reusable cloths, easily wrap able (non-woven) disposable materials can also be used. In internal packing case we recommend, corner of the package materials should be fix with the adhesive tape. In this way the package cannot then open during sterilization and block the inlet and outlet filter holes of the container and raised flow pressure won't damage the container. Because of the problem associated with folding, the use of sterilization paper is not recommended. In order to prevent colors leaching and thereby staining the containers, non-colored materials (or in the case of green or blue cloths, previously washed sheets) should be used.

CAUTION: Never sterilize the container wrapped in additional packaging. Apart from the risk of lack of sterility, the increased flow resistance could impair the sterilization effect (non-sterility) or even destroy the container.

## STERILIZATION OPERATIONAL LIMITS

-In order to ensure that the lid can close properly, sterilization containers must not be filled above the level of the lower ridge of the edge indentation on the container bottoms.

The lid must lie flat on the lower section without being forced and so that it does not wobble even when the clasps are open. It must also be possible to close the clasps without additional pressure on the lid. In the case of instrument sterilization, the load weight (including perforated tray) should not exceed 10 kg for 1/1 size containers. Load weight should be 5kg for 1/2 size containers and smaller loads should be arranged for smaller containers (DIN 58953/9).

- With cloth loads (or folded), the load weight should not exceed 7-8 kg. Make sure that similar textile or cloth loads are placed horizontally in containers (DIN 58953/9). When using internal packaging (non-woven or cloth), care should be taken that the correct closing of the lid is not impeded, for example, by a protruding corner of the packaging.

CAUTION: For example, there is a risk of non-sterility if protruding cloth corners prevent the container from closing correctly.

CAUTION: If the sterilization procedure causes sterilization containers to become deformed in any way, then there is no guarantee of sterility. In such cases, the entire batch must not be used, they should be sterilized again and an investigation should be started to determine the cause.

- In order to prevent damage to the parts of the container and/or its load, we recommend that the container should be transported with its lid closed whenever possible.

## PLACING INTO STERILIZERS

Sterilization containers are made for use in general steam sterilizers (EN 285). Make sure that heavier containers are placed at the bottom of the sterilization chamber, first.

Bahadır aluminum containers were designed that they can be stacked during sterilization. In order to prevent accidents and mechanical damages on the containers it is important to work very carefully with the stacked containers. To prevent condensation collecting on one side (and thus causing drying problems), the containers should be placed horizontally in the sterilizer.

The loading instructions of the sterilizer manufacturer should also be observed.

## DATA CARDS / INDICATORS

We recommend use of information cards with chemical sterilization indicators in the outer holding frame of the containers (DIN 58953/9).

CAUTION: If chemical sterilization indicators are not used, then other organizational measures should be taken to ensure validation of the sterilization and non-sterile-containers being used (released) by mistake.

## AFTER STERILIZATION

To safeguard against accidents (burns, dropping, etc.), containers that are still hot should never be handled with bare hands. The containers should not be cooled to room temperature too rapidly (e.g. do not place on cold surfaces or expose to a cold draught), as excessively rapid external cooling can lead to recondensation of the water vapour inside the container with an unwanted accumulation of condensation.

## STORAGE / TRANSPORTATION

In practice sterility can be maintained for unlimited period with proper packaging, during storage in controlled hospital storage room conditions (temperature, humidity, air filtration etc. controlled). Acceptable storage period should be determined by responsible hygiene personnel. Requirements and suggestions of DIN 58953-9 should be taken under consideration while determining storage time and storage conditions. Depending upon storage duration and conditions, however, external contamination occurs, and this represents a potential risk during subsequent use, transport and aseptic presentation. According to DIN 58953/9 this risk factor can be reduced by the following measures:

- The use of internal packaging.
- Storage under dust protected conditions. The recommendations of DIN 58953-9 on limitation of storage period.
- Containers with internal packaging, protected storage up to:6 months
- Containers with internal packaging, unprotected storage up to:6 weeks
- Containers without internal packaging, protected storage up to:6 weeks
- Containers without internal packaging, unprotected storage use "as soon as possible"

## SPECIAL CASES:

When storing or transporting sterile containers under non-standard conditions (e.g. in case of getting sterilization service for containers from places such as central sterilization departments), then internal packaging and transport packaging should be used to reduce the contamination risks that are associated with outer environment conditions.

## ASEPTIC PRESENTATION

If containers are to be opened after a long period of storage or after storage under non-ideal conditions, then we recommend wiping the unperforated cover with a disinfectant before handling in order to minimize the risk of contamination by air-borne particles.

## CLEANING AND DISINFECTION

Requirements according to DIN 58953/9 :

- Users have to specify by means of a disinfection and cleaning plan, when and how the sterilization containers have to be cleaned and/or disinfected.
  - Containers used for waste disposal have to be cleaned and disinfected each time after use.
  - Cleaning materials should be suitable to available water quality in hand.
- Manual Cleaning**
- Only use neutral cleaners or neutral cleaners and disinfectants for cleaning.
  - Do not use metal brushes or cleaning materials that may cause chemical or physical corruptions.
  - All part must be rinsed with demineralised water with out leaving any stain or residue on them and dried by hand and stored.

## Mechanical Cleaning

- Mechanical cleaning of the containers is to be preferred to manual cleaning.
- Storage under dust protected conditions.
- Cleaning of the containers with machines is only recommended if the washing machine has a special washing program for aluminum containers.
- Only use neutral cleaners or neutral cleaners and disinfectants for cleaning. Do not use any cleaning solutions that contains soda or caustic soda.
- Do not use additional acidic neutralizers.
- Observe the instructions of the manufacturer of neutral cleaners and disinfectants for cleaning aluminum containers.
- Use demineralised water for final rinsing since salt in the water may cause spotting during subsequent sterilizations.
- Cleaning (washing) machine has to be designed for cleaning sterilization containers. This applies in particular to ensure secure replacement in the washing baskets and the arrangement of the spray jets or arms.
- Remove the lids and filter retainers before cleaning the containers and clean them individually.

## OFFICES

**FACTORY AND HEAD OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Organize Sanayi 53300  
SAMSUN/TURKEY  
Tel: 90 (362) 228 70 01 (PBX)  
Fax: 90 (362) 228 70 05

**MARMARA REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Millet Cd. Özbek Süleyman Efendi Sk.  
Araç Apt.No:8/21 Fındıklıcađı / İSTANBUL / TR  
Tel: 90 (212) 632 80 34 - 632 80 35  
Fax: 90 (212) 588 19 76

**EGE REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Mustafa Kemal Paşa Cad.  
Mutlu Apt.No.141 Bornova /İZMİR / TR  
Tel: 90 (232) 343 85 45 - 348 85 46  
Fax: 90 (232) 343 85 48

**AKDENİZ REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Regatbey Mh.Avakut M.Eroglu Sk.  
Berköz Apt.No.2 D:13 Kat:2 ADANA / TR  
Tel: 90 (322) 458 02 72  
Fax: 90 (322) 458 02 92

**KARADENİZ REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
İstiklal Cad. Kaplanpaşa Sk.  
Ali Çetini İğnani No:4/17 SAMSUN / TR  
Tel: 90 (362) 228 65 00 - 228 65 01  
Fax: 90 (362) 228 09 23

**İÇ ANADOLU REGIONAL OFFICE**  
BAHADIR MEDICAL EQUIPMENT  
Talat Paşa Bulvarı  
No.11/13 Kat:ANKARA / TR  
Tel: 90 (312) 310 36 06 - 310 36 07  
Fax: 90 (312) 310 36 08

**EUROPEAN REPRESENTATIVE**  
ERMIS Import -Export  
Möhtinger Straße 1-3  
78532 Tuttlingen/GERMANY  
Phone: +49 7461 16 05 28  
Fax: +49 7461 16 05 98

**FRANCE AGENT**  
SARL SAMMEX  
89, Rue Ernest Renan  
F-51100 Reims/FRANCE  
Phone: +33 326 02 64 30  
Fax: +33 326 02 21 56

**BAHADIR USA CORP.**  
428 Peninsula Blvd.  
Hempstead N.Y 11550 USA  
Tel: +1516 565 3507  
Fax: +1516 565 3794

**AUSTRALIA AGENT**  
TIP TOP FDE  
695 Sydney Rd. Coubrug 3058  
Victoria/AUSTRALIA  
Phone: +613 9350 2377  
Fax: +613 9350 2677

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### CLOGS ARE USED IN;

Hospitals, operation rooms, laboratories, clinics, medical depots, medicine industry, food factories, meat integration foundations, thermal springs, yatchs and where the feet hygiene is concerned.

### CHARACTERISTICS

Manufactured in 7 different collors and in 6 different sizes.  
Different colors are used for determining different branches in an instution.

### CAUTIONS

Do not place the clogs top on each other in auto claves.  
Clogs can be sterilized at 134°C  
Be careful on wet and slippery floors.